
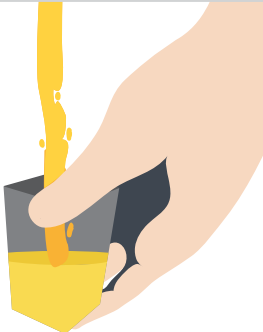
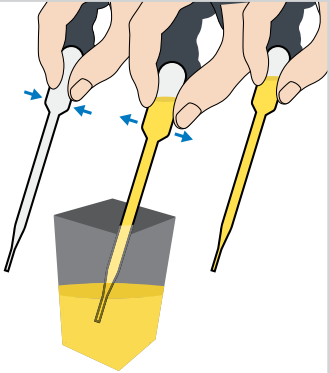
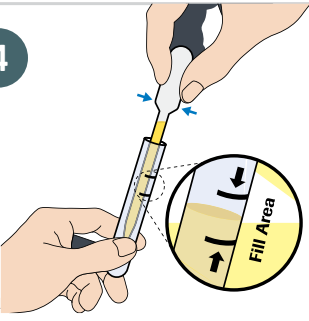
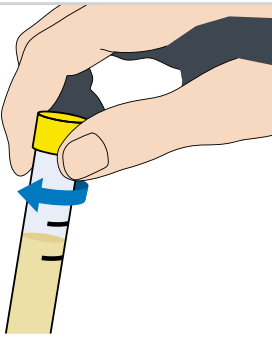
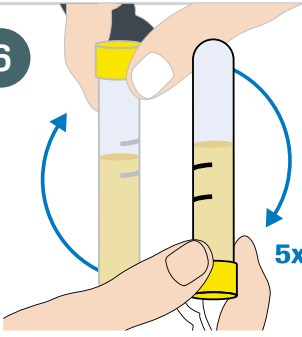
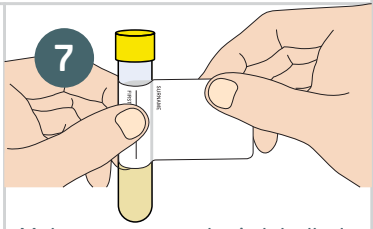

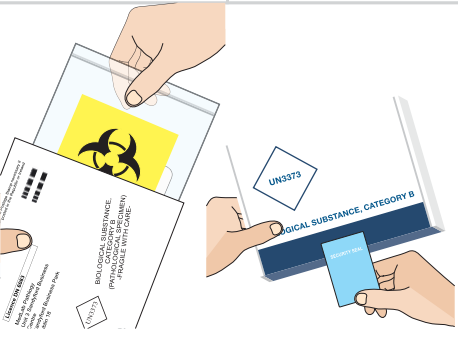


Instructions

URINE SAMPLE COLLECTION



CharterMedical

<p>1</p>  <p>Wash and dry your hands.</p>	<p>2</p>  <p>Pee into the sample urine collection box.</p>	<p>3</p>  <p>Take the provide pipette and use this to transfer the urine into the urine collection vial. This is done by squeezing the end of the pipette when it is inserted into the urine collection box, releasing in order to collect the urine and then squeezing again when the pipette is inserted into the urine collection vial.</p>
<p>4</p>  <p>Fill the vial until the urine level is between the two black horizontal fill lines on the vial.</p>	<p>5</p>  <p>Screw the cap firmly back onto the vial ensuring it is closed securely.</p>	<p>6</p>  <p>Shake the urine vial 5 times</p> <p>7</p>  <p>Make sure your tube is labelled with your details using the label supplied in your collection pack. This is very important as unlabelled samples cannot be processed. Affix the label by placing the tube in the middle of the label and wrapping the label around the tube.</p>
<p>8</p>  <p>Insert sample vial into biohazard bag.</p>	<p>9</p>  <p>Place all your sample(s) in the one Bio-hazard bag provided. Include the request form in the FREEPOST return box. Seal the box with the security label and post in any An-Post post box.</p>	