
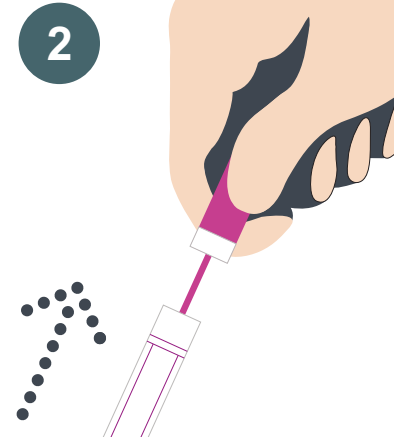
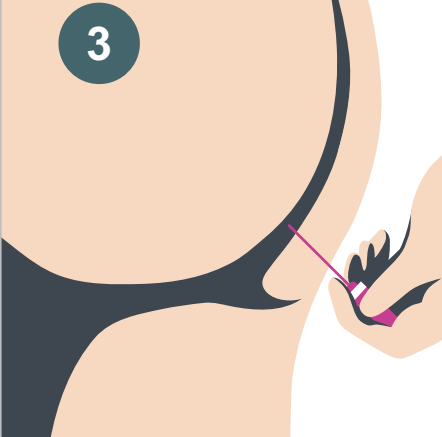
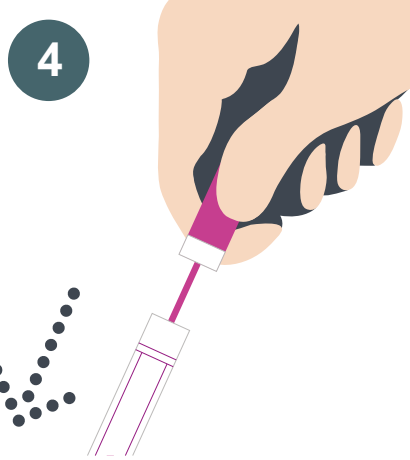
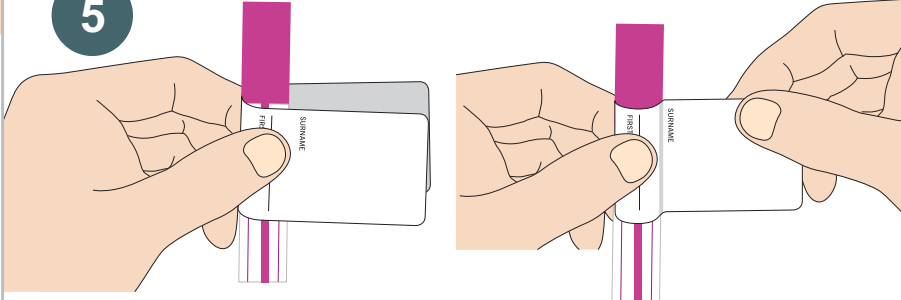

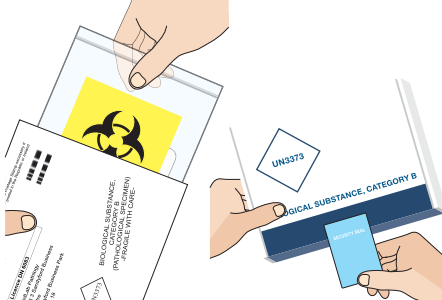




Instructions

RECTAL SWAB SAMPLE

CharterMedical

<p>1</p>  <p>Wash and dry your hands.</p>	<p>2</p>  <p>Twist and pull the swab out of the sample tube.</p>	<p>3</p>  <p>Gently insert the swab into the anal area, twist and remove.</p>
<p>4</p>  <p>Put the swab back into the tube and close firmly.</p>	<p>5</p>  <p>Make sure the tube is labelled with your details using the label supplied in your collection pack. This is very important as unlabelled samples cannot be processed.</p> <p>Affix the label by placing the tube in the middle of the label and wrapping the label around the tube.</p>	
<p>6</p>  <p>Insert sample vial into biohazard bag.</p>	<p>7</p>  <p>Place all your sample(s) in the one Bio-hazard bag provided. Include the request form in the FREEPOST return box. Seal the box with the security label and post in any An-Post post box.</p>	