
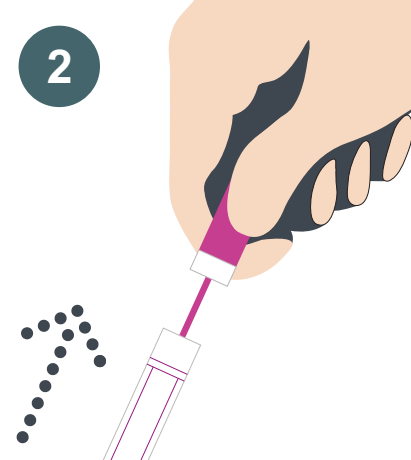
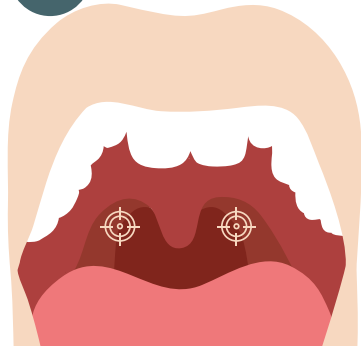
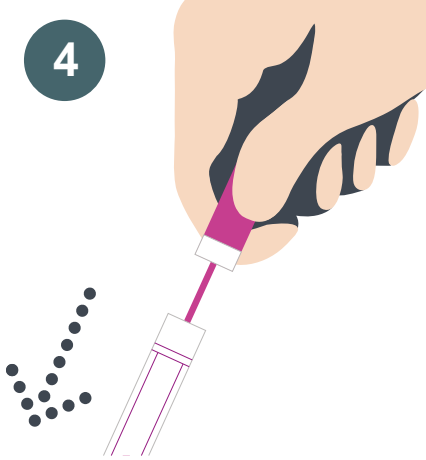
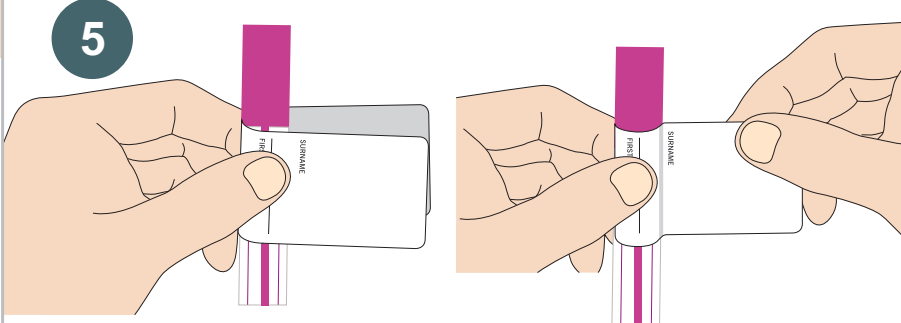

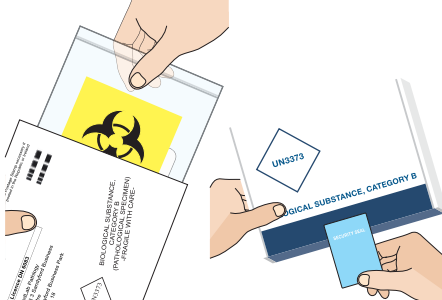




## Instructions

# THROAT SWAB SAMPLE

# CharterMedical

<p><b>1</b></p>  <p>Wash and dry your hands.</p>	<p><b>2</b></p>  <p>Twist and pull the swab out of the sample tube.</p>	<p><b>3</b></p>  <p>Swab around the back of your mouth and throat. You may cough or gag a little.</p>
<p><b>4</b></p>  <p>Put the swab back into the tube and close firmly.</p>	<p><b>5</b></p>  <p>Make sure the tube is labelled with your details using the label supplied in your collection pack. <b>This is very important as unlabelled samples cannot be processed.</b></p> <p>Affix the label by placing the tube in the middle of the label and wrapping the label around the tube.</p>	
<p><b>6</b></p>  <p>Insert sample vial into biohazard bag.</p>	<p><b>7</b></p>  <p>Place all your sample(s) in the one Bio-hazard bag provided. Include the request form in the FREEPOST return box. Seal the box with the security label and post in any An-Post post box.</p>	